


Please pay attention that after install the software we need to install the calibration file . If it was installed , there will be some white lines appear on the image .

Please operate as the following steps to install the calibration .

1) After you insert the CD to your CD-ROM , you will see the autorun interface

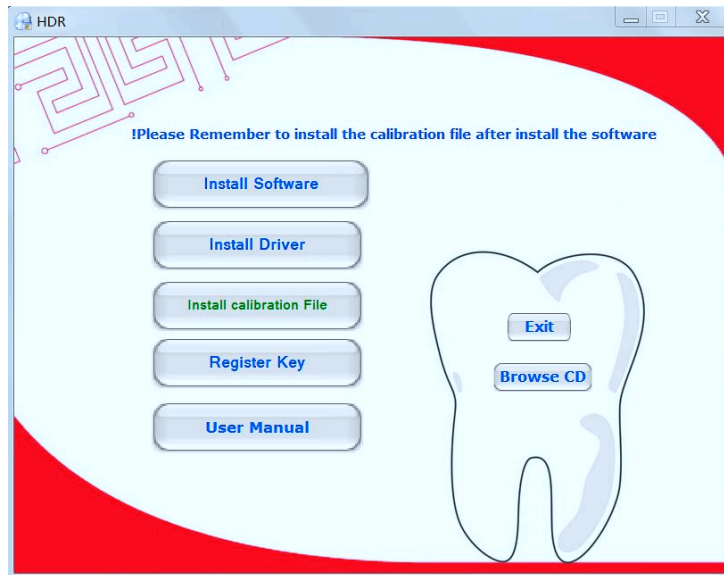

2) Click on Browse CD ,you will see the whole file was open .


3) Please find and open a file named "Calibration file " each sensor has their own unique calibration file . Open the file of calibration you will find three files inside .For example sensor # 9FF5C17 . This sensor calibration files are

- 9FF5C17\_Gain Image
- 9FF5C17\_Offset Image
- 9FF5C17\_Pixel Map

They match with the sensor serial No. Which is attached on the sensor cable .

4) Copy these three files and then paste to the software installed destination file .The default path is Disk C:/program files /HandyDentist .Just copy them to HandyDentist file , then the installation of calibration file finished . If you want to use another sensor into this software , you need to copy that sensor's calibration files to HandyDentist file too .

Before you copy these calibration file , please confirm where you install the software ,during the installation of software there is step to ask you where to install it , please look at the following picture .

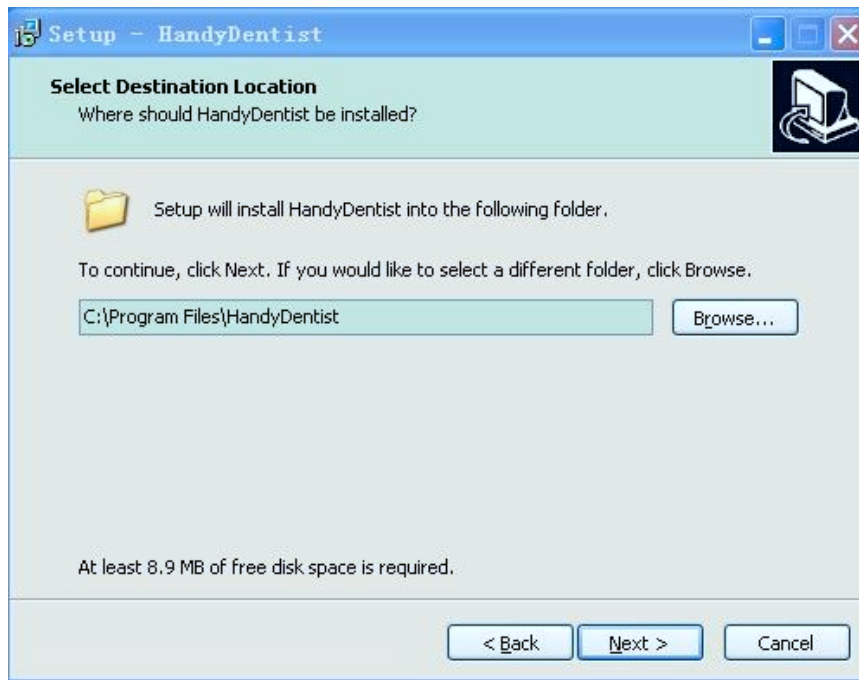

If you click on Next direct and haven't change the the path , it would be the default path of C:/Program Files/HandyDentist